

3M™ Bair Hugger™ Therapy
Temperature Management Unit
Model 775



Clinical Warming With More Control

Quiet. Adjustable. Safe. Effective.
It's everything you want from your
forced-air warming unit...and more.

3M Infection Prevention Solutions

Innovation
On A Mission

3M

Quiet, adjustable warming options for all 3M™ Bair Hugger™ blanket styles

Forced-air warming was introduced to the world in 1987 with the launch of Bair Hugger therapy. Since that time, Bair Hugger warming units and blankets have safely and effectively warmed more than 135 million patients worldwide.

Our latest advancement in forced-air warming – the 3M™ Bair Hugger™ Model 775 temperature management unit – offers two airflow settings, hose-end temp sensing, precise temperature delivery and ease of use from a single, quiet warming unit. It's also compatible with all Bair Hugger blanket styles – pediatric to adult – and 3M™ Bair Paws™ gowns.



Features

- Four temperature settings allow clinicians to deliver effective forced-air patient warming and cooling therapy. The ambient setting can complement your current patient cooling strategies.
- Three temperature sensors and adjustable airflow provide safe, quick temperature response.
- The Model 775 can track and report information on forced-air warming duration, over-temperature conditions and fault codes, all of which are accessible via the front panel – there's no need to open the unit.
- Accessories for the Model 775 warming unit allow the unit to be mounted on a hard surface, an IV pole, bed rail or rolling cart.

Ordering Information

Specifications	Bair Hugger Model 775
Dimensions	13"h × 14"w × 13"d (33h × 35w × 33d cm)
Weight	16 lb (7.3 kg)
Operating Temperatures	High: 43° ± 1.5°C (109.4° ± 2.7°F) Med: 38° ± 1.5°C (100.4° ± 2.7°F) Low: 32° ± 1.5°C (89.6° ± 2.7°F) Ambient: (Room Temperature)
Fan Speeds and Relative Noise	High Fan Setting: 37 CFM (17.5 L/s) 53 dBA Low Fan Setting: 32 CFM (15.1 L/s) 48 dBA
Leakage Current	Meets UL 60601-1 and IEC 60601-1 requirements. CAN/CSA-C22.2, No. 601.1
Safety Systems	Audible and visual alarms; Over-temperature ≤56°C, 53°C ± 3°C typical
Filter	High-efficiency 0.2 µm filter
Device Ratings	110-120 VAC, 50/60 Hz, 11.7 Amperes 220-240 VAC, 50/60 Hz, 7.2 Amperes 100 VAC, 50/60 Hz, 14.5 Amperes
Compatibility	Use only with Bair Hugger blankets and Bair Paws gowns
Accessories	Rolling cart, bed rail mount, accessory handle and teathered clip also available

To order, please contact your 3M Patient Warming representative or call Customer Service at 1-800-733-7775 .



**Arizant Healthcare Inc.,
a 3M company**
10393 West 70th St.
Eden Prairie, MN 55344 USA
Phone 800-733-7775
Fax 800-775-0002
www.bairhugger.com

3M is a trademark of 3M Company, used under license in Canada. BAIR HUGGER, BAIR PAWS and the BAIR HUGGER logo are trademarks of Arizant Healthcare Inc., used under license in Canada. Please recycle. Printed in USA. ©2011 Arizant Healthcare Inc. All rights reserved. 603032D 12/11